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

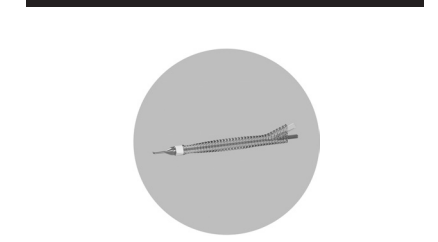
Allegro sp™

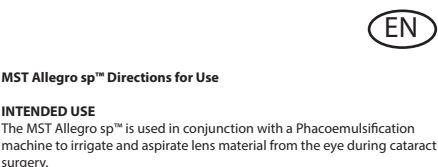67-0045-01 Rev C 6/12/18

MST Allegro sp™ Directions for Use
INTENDED USE
The MST Allegro sp™ is used in conjunction with a Phacoemulsification machine to irrigate and aspirate lens material from the eye during cataract surgery.
CAUTION: The MST Allegro sp™ are provided sterile via irradiation sterilization and are a single use device. Do not attempt to re-sterilize the device. Doing so will damage the tip. Reuse of the product could compromise the integrity of the device and result in injury to the patient.
Warning: Before use check the Allegro sp's tip for damage such as burrs or sharp edges or a dislodged cannula. A damaged tip should be discarded and replaced.
Should damage occur during use discontinue use and replace the handpiece. Use of a damaged tip could result in tearing of the posterior capsule.

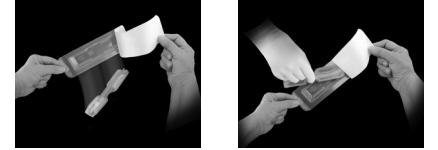DIRECTIONS FOR USE
1. Inspect the device before use to ensure that it is free of damage. Do not attempt to modify the tip.
2. The MST Allegro sp™ comes ready to use. Attach the handpiece to the Phaco machine's tubing set per the user manual provided with the Phaco machine.

3. Removing the Allegro sp from its outer tray can be achieved by either the circulating nurse dropping the inner tray into the sterile field, or the scrub nurse picking the inner tray from it's outer tray.

CAUTION: Never touch the silicone as that will twist /damage the tip and may expose the inner cannula. Exposure of the inner cannula could result in a posterior capsule tear and/or other tissue damage.

STORAGE AND HANDLING The MST Allegro sp™ is labeled for a single use. The tips can be easily damaged and must be inspected prior to use. Do not use the handpieces if damage exists. Avoid contact with other instruments. Do not attempt to repair, bend or otherwise modify the tip. Store at ambient temperature and humidity.

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