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MST Malyugin Ring Directions for Use

INTENDED USE

The MST Malyugin Ring is a manual ophthalmic surgical instrument intended to mechanically stabilize and support ocular tissues/structures during ophthalmic surgery.

The MST Malyugin Ring is indicated to stabilize and support the ocular iris during ophthalmic surgery.

CAUTIONS:

- The device is intended for single use and is sold sterile. Do not attempt to sterilize, reprocess, or reuse the device.**

- Do not move the injector button, retract the ring from the cassette, or separate the injector from the cassette, until ready for use. Retracting the ring into the injector and attempting to re-insert the ring into the cassette can damage the ring.**

- Do not use if package or device appears damaged.**

- Leaving the ring in the inserter may distort the ring. Do not retract the ring into the inserter until it is ready to be placed in the eye.**

- Ensure that other devices do not become entangled in ring scrolls (e.g. IOL haptics, CTRs, etc.).**

- Malyugin Ring is not an implantable device and has not been tested for implantation. Remove the ring from the eye after its use.**

- Reuse of the product could compromise the integrity of the device and result in injury to the patient.**

DIRECTIONS FOR USE

MST Malyugin Rings are shipped ready to use, sterile, in a tray with Tyvek lid.

Ring Placement

- When ready to use, carefully remove the ring assembly from the package, ensuring that the injector button is not moved.
- Retract the injector button, pulling the ring into the cannula.
- Remove the injector from the cassette.
- With viscoelastic in the anterior chamber, and the injector's cannula also in the anterior chamber, move the distal end of the injector cannula to approximately mid