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## **EN** MST Capsule Retractors™ Directions for Use

### DESCRIPTION

MST Capsule Retractors consist of a strand of formed nylon, a silicone disk that helps secure the retractor in place, and a sheath of polypropylene that adds mechanical stability.

### INTENDED USE

The MST Capsule Retractor is a manual ophthalmic surgical instrument intended to mechanically stabilize and support ocular tissues/structures during ophthalmic surgery.

### CAUTIONS


- The Capsule Retractors are sold in a sterile condition and are intended for a single use. Do not attempt to sterilize the device.**
- Do not over tighten any of the retractors. Too much traction on the capsulorhexis could cause it to tear.**
- The MST Capsule Retractors™ are intended for a single use. Do not re-use this product.**
- Ensure that other devices do not become entangled in the retractor (e.g. IOL haptics, CTRs, etc.).**
- Reuse of the product could compromise the integrity of the device and result in injury to the patient.**

### INSTRUCTIONS

Unloading the Retractors:

MST Capsule Retractors are provided in packages of 4, in a thermoformed tray.


- Peel back the lid of a tray, exposing the Retractors in the silicone bar holding them.
- To access the MST Capsule Retractors, turn the tray upside down to drop the bar out of the tray then pinch the bar between your thumb and forefinger to release pressure on the MST Capsule Retractors.
- Grasp the MST Capsule Retractors and gently remove from the bar.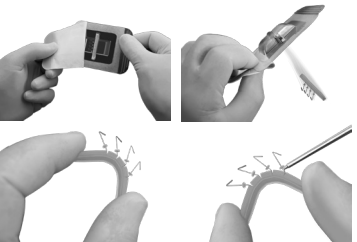

Inserting the Retractors:

Each Retractor is inserted through a clear corneal incision made near the limbus of approximately 0.8mm to 1.2mm.

- Hold the shaft of the Retractor to manipulate the distal hook under the edge of the capsulorhexis and into the capsular bag equator.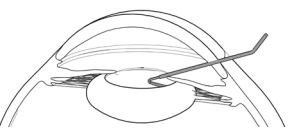

- Once the Retractor is in place, slide the silicone stopper forward to secure the position.

Removing the Retractors:

- Remove the silicone stoppers from the retractors.
- Using forceps or a spatula and a Sinskey hook disengage the retractor and rotate it 90°.
- Carefully pull the retractor into the eye and gently manipulate it through the Phaco incision.
- Repeat this process for the remaining retractors.