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**MST Allegro™ Single Use Silicone I/A Tip**

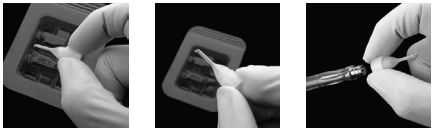
**INTENDED USE**
The Allegro™ I/A System provides the user with a means to aspirate material from, and supply irrigation fluid to the crystalline lens during cataract surgery.
MST Allegro™ Single Use Silicone I/A Tips are used in conjunction with a Phacoemulsification machine and the Allegro™ handpiece to irrigate and aspirate lens material from the eye during cataract surgery.

**CAUTION:** MST Allegro™ Single Use Silicone I/A Tips are supplied in a sterile condition and are a single use device. MST Allegro™ Single Use Silicone I/A Tips are provided sterile via irradiation sterilization. Do not attempt to re-sterilize the device. Doing so will damage the tip. Reuse of the product could compromise the integrity of the device and result in injury to the patient.

#### DIRECTIONS FOR USE

- Inspect the device before each use to ensure that it is free of damage. Do not attempt to modify the tip.
  - Assemble the Allegro™ handpiece and Allegro™ Single Use Silicone I/A tip ensuring that the tip is fastened securely to the handpiece as shown in the figures.
- CAUTION:** Only touch the white base. Twist the tip onto the handle using the white base finger tight.
- CAUTION:** Never touch the silicone as that will twist/damage the tip.
- CAUTION:** Never touch the silicone as that will twist/damage the tip.

**CAUTION:** Only touch the white base. Twist the tip onto the handle using the white base finger tight.

**CAUTION:** Never touch the silicone as that will twist/damage the tip.

#### STORAGE AND HANDLING

The Allegro™ Single Use Silicone I/A Tips are intended for a single use. The tips can be easily damaged and must be inspected prior to each use. Do not use the tip if damage exists. Avoid contact with other instruments. Do not attempt to repair, bend or otherwise modify the tip.

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