Directions For Use

Allegro[™]

Reusable Handpiece for Allegro[™]
Single Use Silicone I/A Tip System

Symbols/ Symbole/ Symboler/ Σύμβολα/ Symboler

| | (EN) | (DE) | (SV) | (NO) | (EL) | (DA) |
|--|---|-----------------------------------|---|--------------------------------|--|--|
| | Non-Sterile | Nicht steril | Osteril | Ikke-steril | Μη αποστειρωμένο | Usteril |
| | Caution | Achtung | Var Försiktig! | Forsiktig | Προσοχή | Forsigtig |
| | Manufacturer | Hersteller | Tillverkare | Produsent | Κατασκευαστής | Producent |
| | Catalog Number | Katalognumme | Katalognummer | Katalognum- mer | Αριθμός καταλόγου | Katalognummer |
| | Batch Code | Chargenbezeich- nung | Satskod | Partikode | Κωδικός παρτίδας | Batchkode |
| | Latex-Free | Latexfrei | Latexfri | Lateksfri | Χωρίς λατέξ | Latex fri |
| | Consult instructions for use | Siehe Gebrauchsanweisung | Se bruksanvisningen | Se bruksanvisningen | Δείτε τις οδηγίες για τη χρήση | Se bruksanvisningen |
| | Authorized Representative in the European Community | Autorisierter Vertreter in der eu | Auktoriserad representant inom europeiska gemenskapen | Autorisert europeirepresentant | Εξουσιοδοτημένος αντιπροσωπός στην Ευρωπαϊκή κοινότητα | Autoriseret representant i det europæiske fællesskab |

Simboli/ Symboles/ Símbolos/ Simboluri/ simboli/ Symbolen

| | (IT) | (FR) | (ES) | (PT) | (FI) | (NL) |
|--|---|--|--|---|--|--|
| | Non sterile | Non stérile | No estéril | Não estéril | Ei-sterilli | Niet-steriel |
| | Attenzione | Attention | Precaución | Cuidado | Varo | Voorzichtig |
| | Produttore | Fabricant | Fabricante | Fabricante | Valmistaja | Fabrikant |
| | Numero di catalogo | Numéro de référence | Número de catálogo | Número de catálogo | Kuvastonu- mero | Catalogus- nummer |
| | Codice lotto | Code de lot | Código de lote | Código do lote | Erikkoodi | Batchcode |
| | Senza lattice | Sans latex | Sin látex | Látex Livre | Lateksiton | Latexvrij |
| | Vedere le istruzioni per l'uso | Voir le mode d'emploi | Véanse las instrucciones de uso | Consultar as instruções de utilização | Kätsko käyttöohjeet | Zie gebruiksaanwijzing |
| | Rappresentante autorizzato per l'unione europea | Représentant agréé dans la communauté européenne | Representante autorizado en la comunidad europea | Representante autorizado na comunidade europeia | Valtuutettu edustaja euroopan yhteisössä | Geautoriseerd vertegenwoordiger in de Europese gemeenschap |

MST Allegro™ I/A Handpiece

Directions for Use

The reusable MST Allegro™ handpiece is only compatible with Allegro™ Single Use Silicone I/A Tips and will not install or function with any other I/A Tip. Attempting to connect with tips other than Allegro™ Single Use Silicone I/A tips may result in damage to the tips or handpiece.

CAUTION: This instrument is packaged in a non-sterile condition. To ensure patient safety, handle the instrument carefully and protect it from damage. Do not use this instrument if it appears damaged in any manner. Do not attempt to alter or repair the instrument. Prior to initial and each subsequent use, wash, flush, rinse, dry, & sterilize the instrument. Improper or inadequate cleaning may result in particulate matter adhering to the instrument and exfoliation into the surgical field. In addition, the function and life expectancy may become compromised.

Intended Use

The MST Allegro™ handpiece combined with Allegro™ tips provides the user with a means to aspirate material, and supply irrigation fluid during cataract surgery.

Directions for Use

The MST Allegro™ handpiece must be sterilized in a steam autoclave prior to each use. The handpiece and tip should not be assembled during autoclaving.

See the directions for use for the Allegro™ Single Use Silicone Tips (67-0069-01) for connection of the tip to the Allegro™ handpiece. Follow the manufacturer's instructions for priming and operating the system.

Cleaning and Sterilization

MST Allegro™ handpieces are designed to be reused after processing –cleaning and sterilization. The following reprocessing steps were validated by MST. Any changes should be validated by the facility performing the processing.

Reprocessing Restrictions

The number of handpiece reprocessing cycles is not limited as long as the post-cleaning and pre-use inspections determine that the product is still functional and shows no signs of damage or corrosion.

Pre-cleaning directly after use

WARNING: To ensure user and patient safety, ensure that no biological contaminants remain on the surface of the handpiece after each use. MST Allegro™ Handpieces must be cleaned directly (without delay) after each use using either the manual or optional automated cleaning process described herein:

The following is the recommended manual cleaning process for the Allegro™ system.

Manual Cleaning

1. DURING THE PROCEDURE

Immediately after use, properly dispose of the Allegro™ Single Use Silicone I/A tip and soak the Allegro™ handpiece in sterile deionized water.

2. POST PROCEDURE CLEANING

This procedure describes the cleaning of the Allegro™ handpiece without the tip attached to the handpiece.

Part I:

- Flush 30cc of deionized water through each lumen.

Part II:

- Sonicate the handpiece in deionized water for 3 minutes.

Part III:

- Flush each lumen of the handpiece with 30cc of air.
- Dry the exterior of the handpiece with a clean lint-free cloth.

NOTE: To avoid damage due to incidental contact, the Allegro™ handpiece should be kept apart from other instruments during cleaning and sterilization.

NOTE: When the Allegro™ handpiece will not be immediately sterilized, it is important to make sure that the instrument is clean and dry. Using a syringe free of water, force 30cc of air through the irrigation line to ensure the line is clear. Repeat this same procedure for the aspiration path.

Automated Cleaning

Thoroughly clean MST Allegro™ handpieces before and between clinical uses: The MST Allegro™ handpiece is compatible with automated cleaning using typical automated cleaning programs designed for thermostable instruments.

MST validated the following automated cleaning cycle using neodisher® MediClean forte alkaline detergent ($10.0 \leq \text{pH} \leq 11$). MST does not recommend the use of enzymatic detergent however if local regulations require their use relative to ophthalmic instruments the devices materials are compatible up to a pH of 11. The enzymatic detergents must be rinsed/neutralized immediately after cleaning/processing per the surgical facility's procedure.

1. Manually clean the handpiece immediately after each use.
 - Flush 30cc of deionized water through each lumen.
2. Program the washer to have the following automated cycle.

Note: Use sterile deionized water only. Prepare the washer and multi-purpose injector per the Operator's Manual.

- Pre-clean 3 minutes then drain.
 - Main wash at a minimum of 55°C for at least 5 minutes with 0.5% cleaning solution then drain.
 - Rinse 3 minutes then drain.
 - Clean 2 minutes then drain.
3. Connect both irrigation aspiration ports to the rinse ports of the cleaning rack and run the automated cycle.

NOTE: Additional rinsing steps will not alter the effectiveness of the validated cycle.

4. When the wash program is completed, visually inspect to ensure the handpiece is clean and dry. Repeat the process as needed.

Sterilization

Do not sterilize prior to performing the cleaning procedure described herein.

1. Sterilize to your facility's standard practice or to industry standards.
2. The following techniques, times and temperatures have been validated by MST:

| Cycle Type | Configuration | Minimum Temperature | Minimum Exposure Time (minutes) | Minimum Drying Time (minutes) |
|------------|---------------|---------------------|---------------------------------|-------------------------------|
| Gravity | Un-wrapped | 132° (270°) | 10 | NA |
| Gravity | Wrapped | 132° (270°) | 10 | 15 |
| Pre-vacuum | Un-wrapped | 132° (270°) | 4 | NA |
| Pre-vacuum | Un-wrapped | 135° (275°) | 3 | NA |
| Pre-vacuum | Wrapped | 135° (275°) | 3 | 16 |
| Pre-vacuum | Wrapped | 132° (270°) | 4 | 20 |

Maintenance

The MST Allegro™ handpiece requires regular cleaning and sterilization after each use. The MST Allegro™ handpiece should be inspected for signs of damage or build up of cortical material in the fluid pathways at least once per day of use.

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MicroSurgical Technology Inc
MDSS GmbH